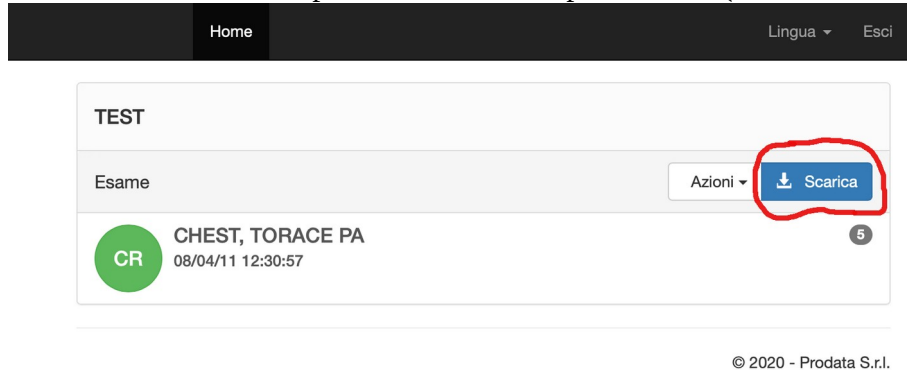
Guida rapida MPWEB

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
For quick access to the original images and / or to receive the data package that can be used at a specialist office, **the end user must use the blue "DOWNLOAD"** button as per the attached image.


This operation allows you to recover within a single ZIP file:

- Original images of the radiological examination (Dicom images)
- Report in PDF format
- Professional multi-platform viewer (Windows, Mac and Linux)

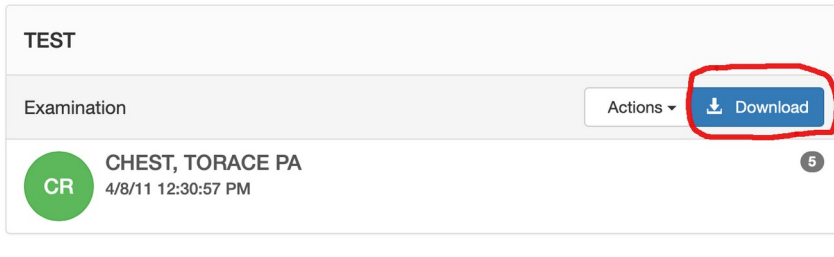
© 2020 - Prodata S.r.l.

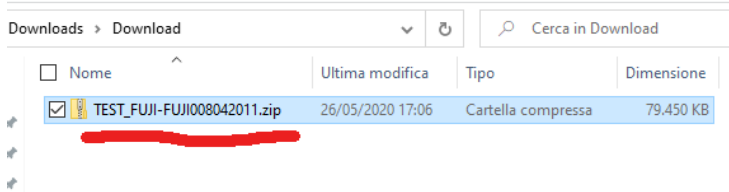
The compressed file corresponding to the contents of the CD is saved in the Download folder.

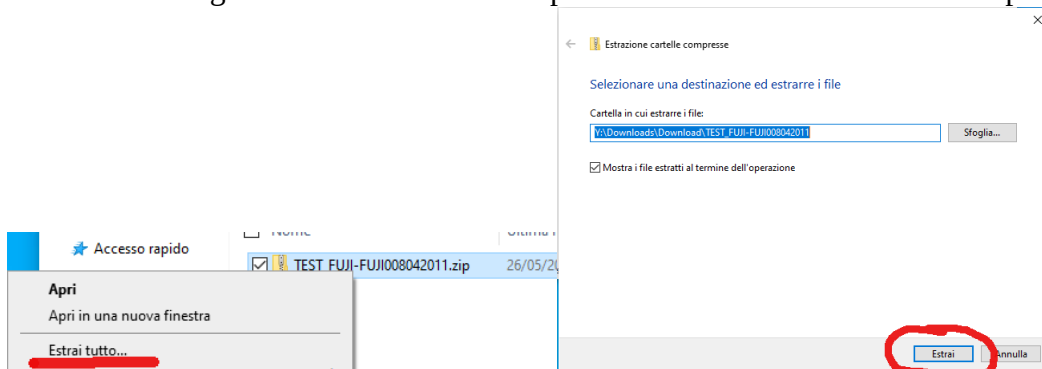It is necessary to unzip the entire zip file into a folder. Follow this procedure: click with the right mouse button on the zip file and choose the "Extract all" option

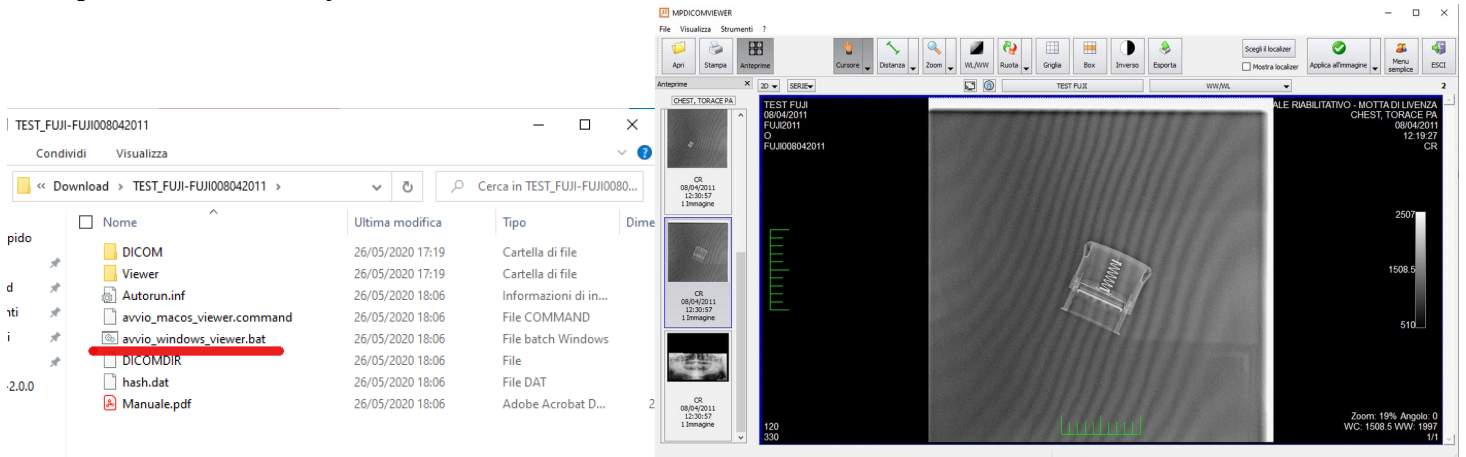From the folder you can start the viewer by clicking on the file relating to the path of the operating system used for example. For Windows systems it is **avvio_windows_viewer.bat**

DICOM viewer interface from where it will be possible to apply a series of functions to process the images. The operating manual of this program is resident in the system and will be opened by clicking on the menu "?"

You can copy this content onto a USB stick and view it on another computer